

Vitaport 4 Patient Instruction sheet

Not allowed:

- Use in combination with cardiac pacemakers or other stimulators.
- Use in shower, bath or sink.
- To immerse the SpO2 probe in liquid.
- Switching off the recorder.
- Pressing the pouch/event button for longer than 3 sec.
- Removing the memory card.
- Disconnecting any cable.
- Connecting anything to any connector.
- Move within 1 meter of other electric devices, such as TV, microwaves, mobile phones etc.
- Spilling water or other liquid onto any part of the Vitaport.
- Removing or pulling any sensor or electrode.

Allowed:

- Pressing the event button on instructed events not longer than 3 sec.

Contact lab when:

- There is damage to, or unexpected behaviour of, the recorder.
- Dropped and broken.
- Unexpected sounds generated by recorder.

Call: _____